


# PiezoLED™

## Prophylaxis and Implant Cleaning

KaVo PiezoLED tips overview

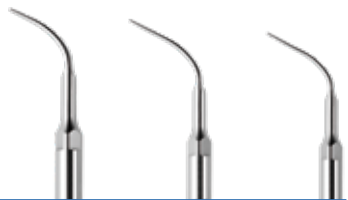

# PROPHYLAXIS



## PIEZO Scaler tips

For the removal of supra- and subgingival deposits in all quadrants

Application in the dental interstices, in the sulcus area and on root surfaces

Effective and time-saving calculus removal and prophylactic treatment

Also suitable for **PIEZOssoft**



## PIEZO Scaler tip 201

Universal scaling tip

- For the removal of supragingival calculus in all quadrants
- Work from the incisal edge to the sulcus
- Apply tangentially to tooth
- Only use the first 2-3 mm of the tip



Mat.-No. 1.007.4024

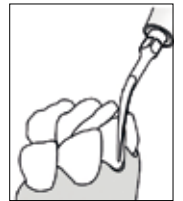
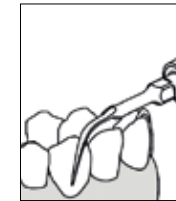


## PIEZO Scaler tip 202

- Removal of supra- and subgingival calculus in all quadrants up to a pocket depth of 4 mm
- Especially suitable for dental interstices and the sulcus area
- Apply tangentially to tooth
- Only use the first 2-3 mm of the tip



Mat.-No. 1.007.4026

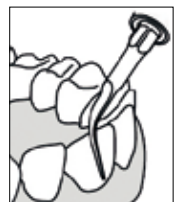
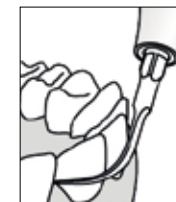


# PROPHYLAXIS

## PIEZO Scaler tip 203

Fine paro tip

- For removal of subgingival deposits on root surfaces
- For deep periodontal pockets and periodontal preventative treatments/checkups
- Apply tangentially to tooth
- Only use the first 2-3 mm of the tip
- Gentle treatment (also applicable for children)

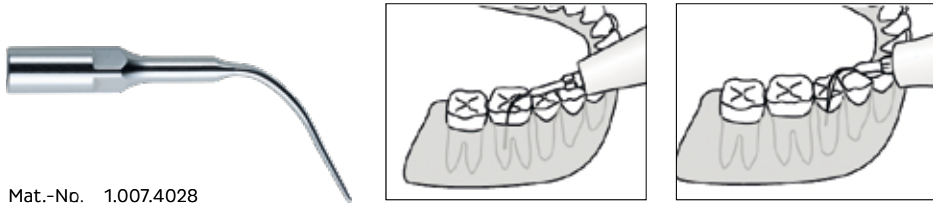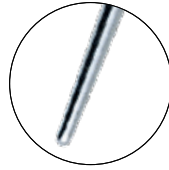

Mat.-No. 1.007.4028



# IMPLANT CLEANING



## PIEZO Implant tip set

For the efficient cleaning of implants

Metal-free tip prevents the damage of the implant neck



Very smooth surfaces with roughening

## PIEZO Implant tip set

- For removing plaque and calculus from implant surfaces and metal or ceramic restorations
- For the efficient and gentle cleaning of surfaces of prostheses
- Metal-free tip
- 50 applications per set




Mat.-No. 1.007.4008  
Refill tip set: 1.007.4014

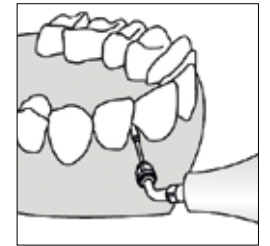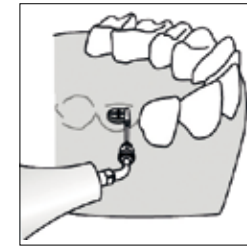

Material	Description	PIEZO soft	Piezo LED	Content
<b>PIEZO Scaler tips</b>				
1.007.4004	PIEZO Scaler tip set	●	●	3 PIEZO Scaler tips (No.s 201, 202, 203 – one of each – with torque wrench) Six-piece Steribox Wear and tear measuring card
1.007.4024	PIEZO Scaler tip 201	●	●	
1.007.4026	PIEZO Scaler tip 202	●	●	
1.007.4028	PIEZO Scaler tip 203	●	●	
<b>PIEZO Implant tips</b>				
1.007.4008	PIEZO Implant tip set		●	Holder for PIEZO Implant tips Wrench for PIEZO Implant tip holder 4 attachments
1.007.4014	PIEZO Implant refill tip set		●	4 attachments

1.500.8099 en 04/21 REV1 © Copyright kaVo Dental GmbH.

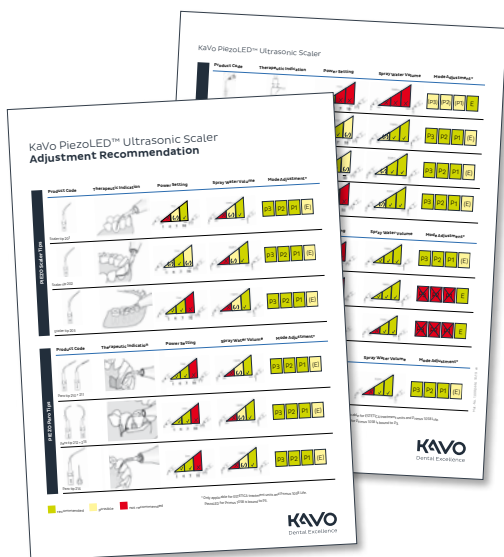

#### Note PiezoLED:

Please observe our setting recommendations for the tips.